

# Measuring Guidelines for

# CUSTOM STOCKINGS - SEAMLESS OR FLAT KNIT

LEFT leg circumference	WHERE TO MEASURE	RIGHT leg circumference	LEG LENGTH from body part to floor	PANTY TOP LENGTH
cm	Waist <b>m</b>	<b>m</b>	<b>m</b> to floor cm	
cm	Hips <b>l</b>	<b>l</b>	<b>l</b> to floor cm	FRONT cm
cm	Buttocks <b>k</b>	<b>k</b>	<b>k</b> to floor cm	BACK cm
cm	<b>g</b>	<b>g</b>	<b>g</b> to floor cm	
cm	<b>f</b>	<b>f</b>	<b>f</b> to floor cm	
cm	<b>e</b>	<b>e</b>	<b>e</b> to floor cm	
cm	<b>d</b>	<b>d</b>	<b>d</b> to floor cm	
cm	<b>c</b>	<b>c</b>	<b>c</b> to floor cm	
cm	<b>b1</b>	<b>b1</b>	<b>b1</b> to floor cm	
cm	<b>b</b>	<b>b</b>	<b>b</b> to floor cm	
cm	<b>a1</b>	<b>a1</b>		
cm	<b>h</b>	<b>h</b>		
cm	<b>A - Closed Toe</b>	<b>A - Open Toe</b>		
cm				

- m** measurement at waist
- l** measurement just above pelvic bone
- k** measurement at widest part of hips
- g** measurement at top of thigh at gluteal fold
- f** measurement at mid thigh
- e** measurement slightly above knee
- d** measurement slightly below knee
- c** measurement at widest part of calf
- b1** measurement between ankle and widest part of calf
- b** measurement just above ankle bone
- a1** measurement circumference of ball of the foot
- h** measurement diagonally around heel over widest part of top of ankle
- A** measurement from heel to toe for Closed-toe stockings (or) from heel to ball of foot for Open-toe stockings

## HOW TO MEASURE FOR MEDIVEN CUSTOM-MADE MEDICAL COMPRESSION STOCKINGS

1. First, measure the circumferences around the specified body parts.
2. Next, measure the height from each specified circumference to the floor.

Over 98% of patients can be fitted precisely to their medical needs in Mediven ready-made stockings. However, for patients requiring a non-standard size, Medi offers a full line of custom-made seamless and flat knit stockings.

Incorporating up to 24 measurements of circumference and length, Mediven custom stockings are precisely engineered to fit each patient's individual profile – from waist, hips, and thighs, to calves, ankles, and feet. And, Medi sales and service representatives are available to provide technical assistance, telephone support, and even in-store training on how to measure and fit Mediven products.

Just like Mediven standard stockings, custom-made products are seamless, attractive, comfortable, and machine washable and dryable. Custom stockings are available in 20-30; 30-40; 40-50 mmHg.

**Always use your Mediven custom stocking order form to order**

### Key for Chart

\* Height measurement is from each marked body location to floor